



PRODUCT CATALOG



THE CAREALINE SLEEVE


A PROACTIVE APPROACH TO SECURING PICC LINES

The CareAline Sleeve is designed to keep lines off the skin and covered, as well as stabilized during use. The sleeve secures the line between two layers of cotton/spandex fabric, and includes a pocket area to hold caps and extensions when they are not in use. The CareAline Sleeve can be used for PICC lines and other catheters and tubes that come out of the arm or leg.

| Product Code | Size | Inches | Centimeters |
|--------------|------|--------|-------------|
| CAL-1S | 1 | 4-6 | 10-15 |
| CAL-2S | 2 | 6-8 | 15-20 |
| CAL-3S | 3 | 8-10 | 20-25 |
| CAL-4S | 4 | 9-11 | 23-28 |
| CAL-5S | 5 | 11-13 | 28-33 |
| CAL-6S | 6 | 13-16 | 33-40 |
| CAL-7S | 7 | 15-17 | 38-43 |




WEAR AND CARE INSTRUCTIONS

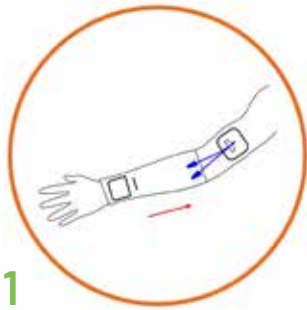

1

With the sleeve partially or fully unfolded, and the buttonhole and pocket lined up with the tubing insertion site, pull the sleeve onto the arm.

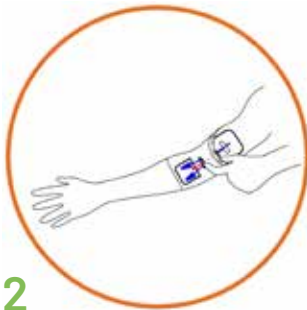


2

With the top of the sleeve at the bottom of your insertion site, lift the top layer of the folded sleeve and insert your finger into the buttonhole pinching the fabric together. Feed the line(s) under the top of the sleeve and out through the buttonhole.



3

Lift the pinched fabric over the insertion site, then place the cap(s) in the pocket.

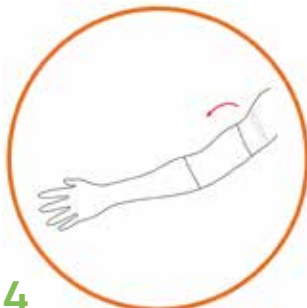

4

Fold the top of the sleeve down over the second layer, covering the buttonhole, lines and pocket.

Access the line as indicated while keeping the sleeve in place and feel secure knowing that your sleeve is perfectly safe to wear during your scans.

WASHING

- All of our products can be machine washed on any cycle. We recommend machine drying on a medium heat cycle.
- When in the hospital, you can wash in the sink and lay flat to dry when necessary. Avoid stretching fabric when wet. Do not wring or hang to dry.
- We recommend washing every 1-3 days and when soiled.

Instructional Videos for application, use and removal can be found at:
[CareAline.com/vascularaccessproducts](https://www.carealine.com/vascularaccessproducts)





THE CAREALINE WRAP


A PROACTIVE APPROACH TO SECURING CENTRAL LINES AND FEEDING TUBES

The CareAline Wrap is designed to keep lines off the skin and covered, as well as stabilized during use. The wrap secures the line between two layers of cotton/spandex fabric, and includes a pocket area to hold caps and extensions when they are not in use. The CareAline Wrap can be used for central lines (Broviac, Hickman, Groshong, etc.), G-tubes, PEG tubes, and other lines and catheters that come out of the torso.

| Product Code | Size | Inches | Centimeters |
|--------------|------|--------|-------------|
| CAL-1W | 1 | 16-19 | 40-49 |
| CAL-2W | 2 | 20-25 | 51-64 |
| CAL-3W | 3 | 24-30 | 61-76 |
| CAL-4W | 4 | 29-35 | 74-89 |
| CAL-5W | 5 | 34-40 | 86-102 |
| CAL-6W | 6 | 39-45 | 99-114 |
| CAL-7W | 7 | 44-50 | 112-127 |




WEAR AND CARE INSTRUCTIONS

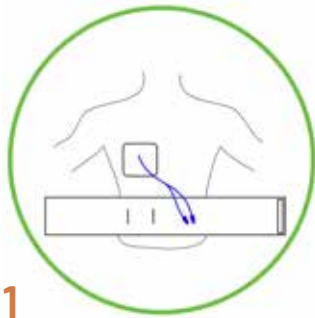


1

Leaving the sterile dressing in place, lay the wrap in front of the patient with the label facing down.

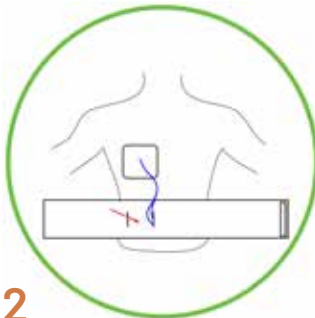

2

Pass the line through one of the two button holes. Use the first hole for longer lines and the second for shorter lines.

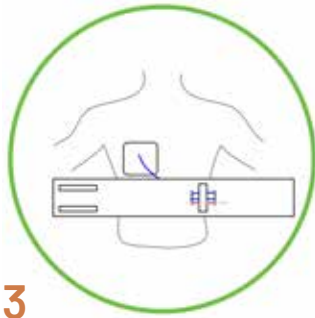

3

Thread the line between the two layers of fabric and exit through the first buttonhole on the opposite side of the wrap. Slide the cap(s) under the label and into the second buttonhole.



4

Stretch the wrap around the body and secure with Velcro.

Access the line as indicated while keeping the wrap in place and feel secure knowing that the wrap is perfectly safe to wear during scans.

WASHING

- All of our products can be machine washed on any cycle. We recommend machine drying on a medium heat cycle. Secure the Velcro before washing to protect other clothing.
- When in the hospital, you can wash in the sink and lay flat to dry when necessary. Avoid stretching fabric when wet. Do not wring or hang to dry.
- We recommend washing every 1-3 days and when soiled.

Instructional Videos for application, use and removal can be found at:
[CareAline.com/vascularaccessproducts](https://www.carealine.com/vascularaccessproducts)





THE CAREALINE PRO+ SLEEVE


PROFESSIONAL PROACTIVE PROTECTION

The CareAline PRO+ Sleeve is designed to keep lines off the skin and covered, as well as stabilized during use. The sleeve secures the line between two layers of cotton/spandex fabric, and includes a pocket area to hold caps and extensions when they are not in use. The PRO+ Sleeve can be used for PICC lines and other catheters and tubes that come out of the arm or leg. It can be placed and removed without disconnecting the line, making them ideal for inpatient use and patients on continuous infusions.

| Product Code | Size | Inches | Centimeters |
|--------------|------|--------|-------------|
| PRO-1S | 1 | 4-6 | 10-15 |
| PRO-2S | 2 | 6-8 | 15-20 |
| PRO-3S | 3 | 8-10 | 20-25 |
| PRO-4S | 4 | 9-11 | 23-28 |
| PRO-5S | 5 | 11-13 | 28-33 |
| PRO-6S | 6 | 13-16 | 33-40 |
| PRO-7S | 7 | 15-17 | 38-43 |




PRO+ WEAR AND CARE INSTRUCTIONS

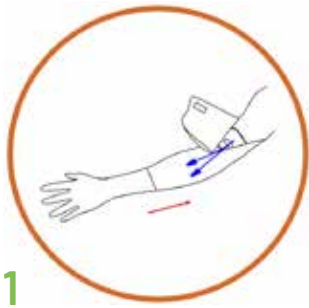


1

Leaving the sterile dressing in place, with the sleeve partially or fully unfolded, reveal the pocket flap and slit. Line the slit up with the tubing insertion site and pull the sleeve onto your arm.

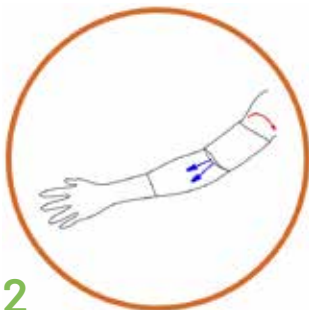


2

Slide the sleeve up so the lines go on top of the fabric, and the bottom of the slit is at the base of the dressing. Stretch the pocket/flap and attach the Velcro closure.

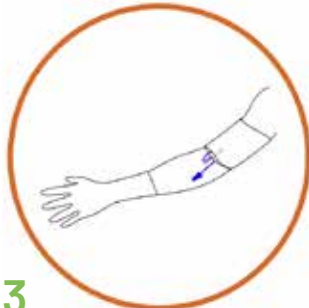

3

Tuck the cap(s)/needleless connector(s) into the pocket on the flap if not in use. If line is connected, leave connected and out of pocket until disconnected.

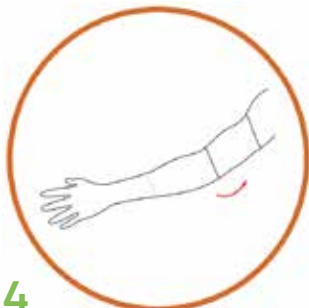

4

Fold the bottom (sewn) edge of the sleeve up toward the top, covering the pocket/flap and lines.

Access the line as indicated while keeping the sleeve in place by rolling down the top layer and removing the line from the pocket. Feel secure knowing that your sleeve is safe to wear during your scans.

WASHING

- All of our products can be machine washed on any cycle. We recommend machine drying on a medium heat cycle. Secure the Velcro before washing to protect other clothing.
- When in the hospital, you can wash in the sink and lay flat to dry when necessary. Avoid stretching fabric when wet. Do not wring or hang to dry.
- We recommend washing every 1-3 days and when soiled.

Instructional Videos for application, use and removal can be found at:
[CareAline.com/vascularaccessproducts](https://www.carealine.com/vascularaccessproducts)





THE CAREALINE PRO+ WRAP


PROFESSIONAL PROACTIVE PROTECTION

The CareAline Wrap is designed to keep lines off the skin and covered, as well as stabilized during use. The wrap secures the line between two layers of cotton/spandex fabric, and includes a pocket area to hold caps and extensions when they are not in use. The CareAline Wrap can be used for central lines (Broviac, Hickman, Groshong, etc.), G-tubes, PEG tubes, and other lines and catheters that come out of the torso. It can be placed and removed without disconnecting the line, making them ideal for inpatient use and patients on continuous infusions.

| Product Code | Size | Inches | Centimeters |
|--------------|------|--------|-------------|
| PRO-1W | 1 | 16-19 | 40-49 |
| PRO-2W | 2 | 20-25 | 51-64 |
| PRO-3W | 3 | 24-30 | 61-76 |
| PRO-4W | 4 | 29-35 | 74-89 |
| PRO-5W | 5 | 34-40 | 86-102 |
| PRO-6W | 6 | 39-45 | 99-114 |
| PRO-7W | 7 | 44-50 | 112-127 |




PRO+ WEAR AND CARE INSTRUCTIONS

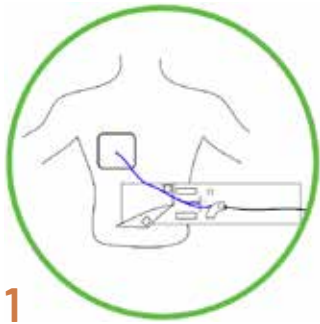


1

Leaving the sterile dressing in place, Lay the wrap in front of you with the label facing up. Open the Velcro flap.*

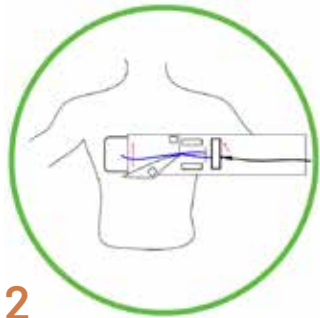


2

Insert the cap(s)/needleless connector(s) into the buttonhole. If the line is connected, undo the Velcro on the label and place the line between the label and the wrap, and replace Velcro.

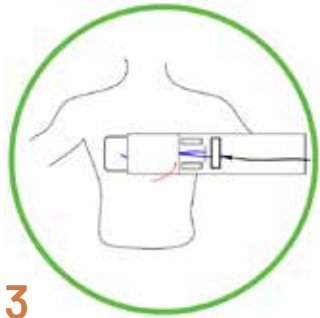

3

Slide the single layer of fabric of the wrap between the line and the body, fold up the flap and secure the Velcro over the line.

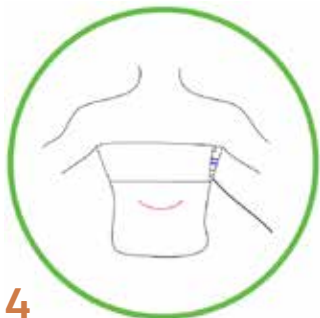

4

Stretch the wrap around the body and secure with Velcro.

Access the line as indicated while keeping the wrap in place by removing the cap(s)/needleless connector(s) from the buttonhole, and then putting the Velcro label around the connected line. Feel secure knowing that your wrap is safe to wear during your scans.

WASHING

- All of our products can be machine washed on any cycle. We recommend machine drying on a medium heat cycle. Secure the Velcro before washing to protect other clothing.
- When in the hospital, you can wash in the sink and lay flat to dry when necessary. Avoid stretching fabric when wet. Do not wring or hang to dry.
- We recommend washing every 1-3 days and when soiled.

Instructional Videos for application, use and removal can be found at:
CareAline.com/vascularaccessproducts

*The first time you use your wrap, measure how long your line is from where it comes out from under the dressing to the base of the cap/needleless connector. Trim your wrap (away from your line) using a pair of sharp scissors so that the measurement from the buttonhole to the edge of the wrap is slightly shorter than this measurement.



CLINICALLY PROVEN TO REDUCE COMPLICATIONS

CareAline Vascular Access Securement products have been clinically evaluated by top hospitals in the US. They have been proven to reduce the risk of three major line complications - Dislodgment, Breakage and infection. To find out more about the clinical evaluations completed to date, visit our website: CareAline.com/clinicalevidencedetails/

2016, RAINBOW BABIES CHILDREN'S HOSPITAL

In 2016, Rainbow Babies Children's Hospital recognized a need to decrease central line related infections. They implemented a plan that utilized CareAline wraps in a CLABSI prevention maintenance bundle. A collaborative team including infection control, central line and quality experts formed a taskforce to identify gaps in the infection control process of central line care and maintenance.

CONCLUSION

The goal was to decrease central line related infections by 40% over a period of one year and this was accomplished in part by including CareAline wraps and sleeves in line maintenance bundles.

2016, C.S.MOTT CHILDREN'S HOSPITAL

A 2016 study done by C.S. Mott Children's Hospital and the University of Michigan revealed that forty-four percent of CVC replacements were in pediatric patients less than four years of age. A pilot study was initiated with CareAline. In an effort to prevent additional replacement surgeries, the central line wrap/sleeve was introduced to secure the lines, in addition to a sterile dressing.

CONCLUSION

Utilization of the CVC line wrap/sleeve has demonstrated a decrease in CVC breakages in pediatric oncology patients. Value analysis partners agreed upon stocking CareAline sleeves and wraps in all sizes throughout C.S. Motts and they are now available hospital-wide.

2014, CHILDREN'S HOSPITAL OF COLORADO


In 2014 the Children's Hospital of Colorado carried out a proof-of-concept analysis within the Center for Cancer and Blood Disorders (CCBD), based on the Anschutz Medical Campus.

CONCLUSION


Accidental line removals went from 23% to zero while patients were wearing a CareAline product.




ALTERNATE WAYS TO WEAR WRAP AND SLEEVE

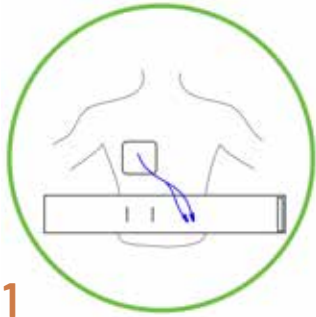


Leaving the sterile dressing in place, lay the wrap in front of you with the label facing down.

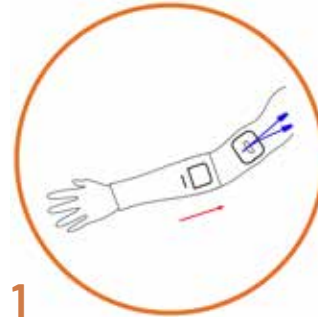


Leaving the sterile dressing in place with the sleeve partially or fully unfolded, reveal the pocket and buttonhole. If your lines point toward your shoulder, you can pull the sleeve on with the pocket at the top.

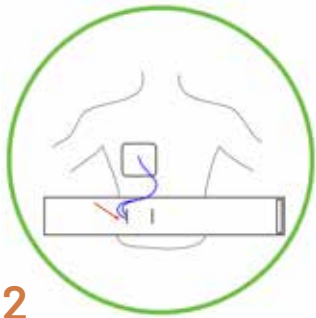


Pass the line through one of the two button holes - this picture shows the hole for a long line.

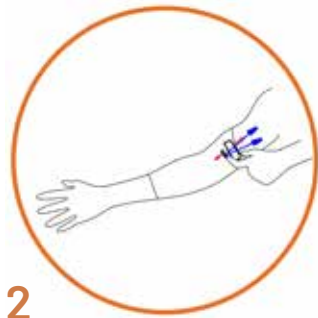

With the edge of the sleeve at the bottom of your insertion site, lift the edge of the sleeve and insert your finger into the buttonhole pinching the fabric together. Feed the line(s) under the edge of the sleeve and out through the buttonhole.

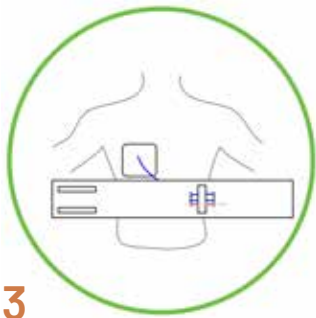


Feed the line between the two layers of fabric and exit through the first buttonhole on the opposite side of the wrap. Slide the cap(s) under the label and into the second buttonhole.

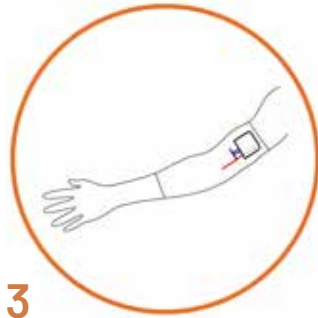

Lift the pinched fabric over the insertion site, then place the cap(s) in the pocket.



Stretch the wrap around the body and secure with velcro. You can choose how you would like to wear your wrap around your body.

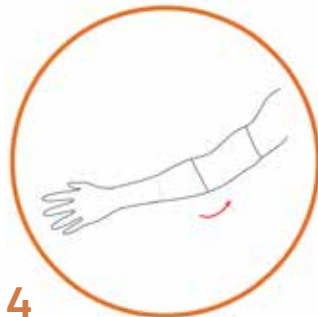

Fold the top of the sleeve down over the second layer, covering the buttonhole, lines and pocket.

Access the line as indicated while keeping the wrap in place and feel secure knowing that the wrap is perfectly safe to wear during scans.

Access the line as indicated while keeping the sleeve in place and feel secure knowing that the sleeve is perfectly safe to wear during scans.



CONTACT INFO

Phone: 617-548-7419
Email: help@carealine.com
Fax: 888-511-0484
CareAline Products
17 Conant St #61
Danvers, MA 01923
USA

July 2022